

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
SHUTTLE FERRY[®] II

Suture Passer for Rotator cuff repair



9


Grab the suture loop and retrieve the suture passer slowly.



Repeat step 5-7 until preparation is finished.

10

When preparation is finished, take the needle out of the suture passer device and dispose the needle properly (single use)

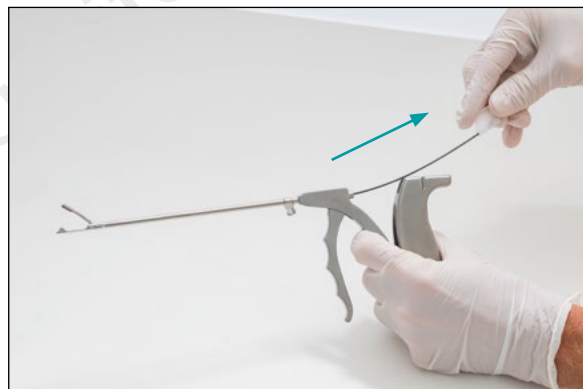

Ordering information

Catalog No.	Product Description
BAS-9169	SHUTTLE FERRY [®] II Suture Passer for Rotator cuff repair
BAS-9170	Needle for SHUTTLE FERRY [®] II- suture passer (Nitinol)



BIOTEK - Chetan Meditech Pvt. Ltd.

Opp. V. S. Hospital, Ellisbridge, Ahmedabad-380 006, Gujarat, INDIA.
Phone: +91 79 26578092; 26577639 & 40098384
Email: contact@biotekortho.com Website: www.biotekortho.com

An ISO 13485 : 2016 Company
All Implants specified in the catalogue are CE certified

